
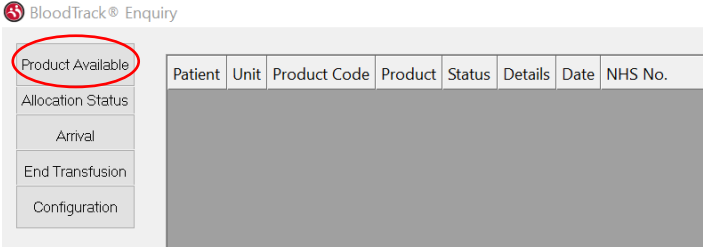
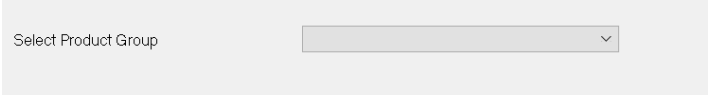
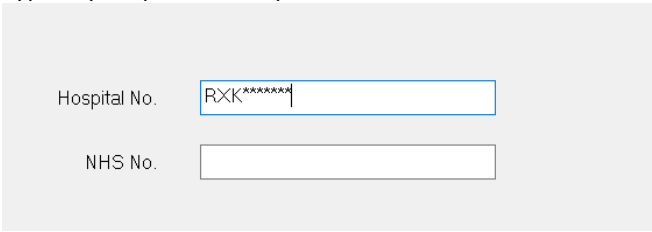
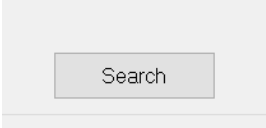
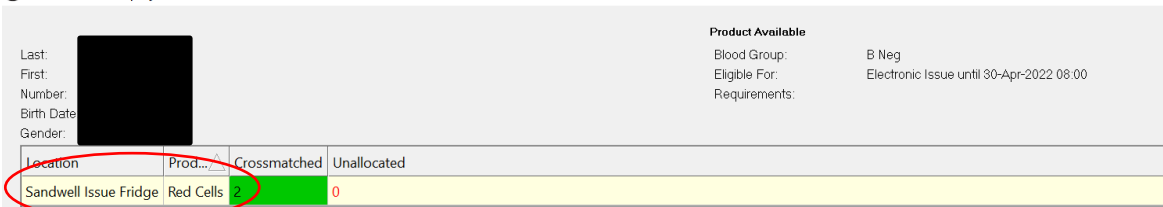






How to Use BloodTrack Ward Enquiry




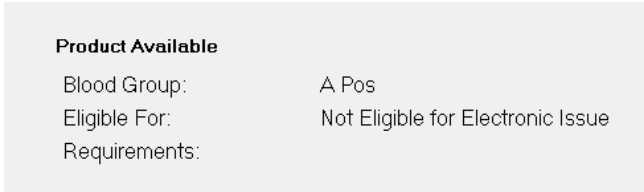
BloodTrack Ward Enquiry is a desktop based application which allows clinical staff to view what blood components are available for collection for their patients.

Step	Action
• How To Use BloodTrack Ward Enquiry	
1	<p>Double left click on the BloodTrack Ward Enquiry icon on the computer desktop screen to open the application</p> 
2	<p>Click on 'Product Available'</p> 
3	<p>Click on the drop down menu and select 'All Products'</p>  <p>This will allow you to see all products available for you patient</p>
4	<p>Type in your patients hospital number</p> 
5	<p>Then press 'Search'</p> 
6	<p>The next screen shows if the blood component requested has been issued for collection and its specific location</p>  <p>This shows that 2 units of red cells have been issued into the Sandwell Issue fridge for this patient</p>

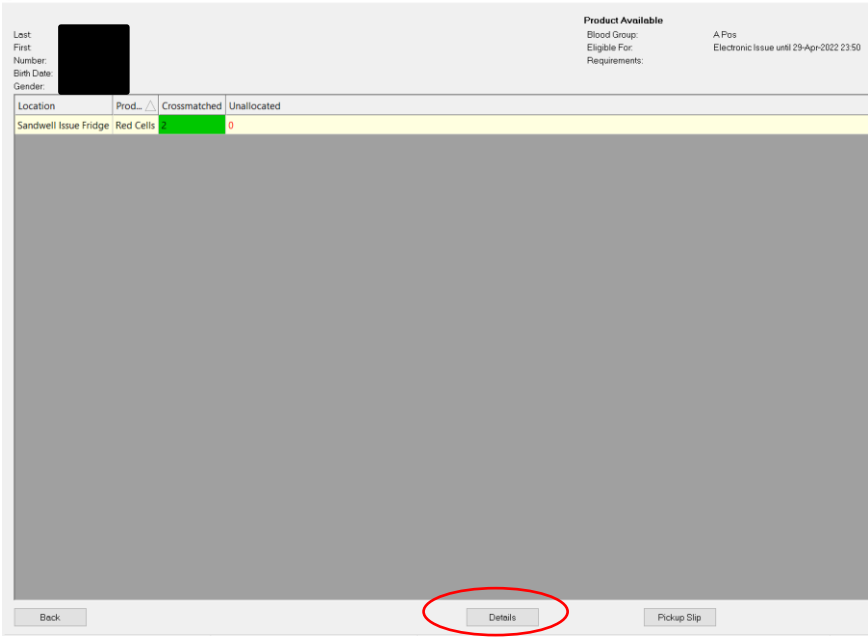
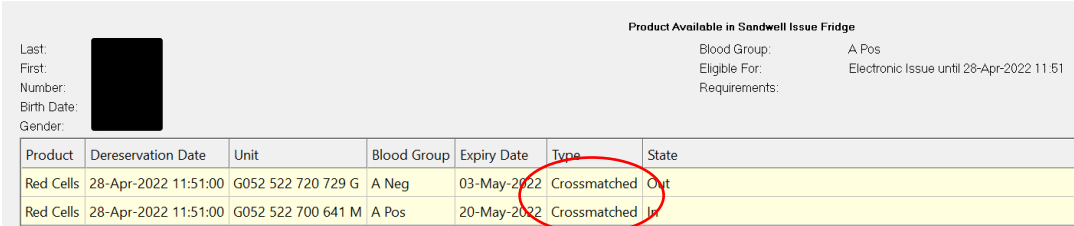


Step	Action																					
7	<p>Click on 'Details' to see the status of the crossmatched units</p>  <p>The screenshot shows a patient enquiry page for 'Product Available'. At the bottom, the 'Details' button is circled in red.</p>																					
8	<p>This shows that 1 unit is 'out' and has already been collected for this patient and another is 'In' and available for collection</p>  <p>The screenshot shows a table of units with the following data:</p> <table border="1"><thead><tr><th>Product</th><th>Dereservation Date</th><th>Unit</th><th>Blood Group</th><th>Expiry Date</th><th>Type</th><th>State</th></tr></thead><tbody><tr><td>Red Cells</td><td>28-Apr-2022 11:51:00</td><td>G052 522 720 729 G</td><td>A Neg</td><td>03-May-2022</td><td>Crossmatched</td><td>Out</td></tr><tr><td>Red Cells</td><td>28-Apr-2022 11:51:00</td><td>G052 522 700 641 M</td><td>A Pos</td><td>20-May-2022</td><td>Crossmatched</td><td>In</td></tr></tbody></table> <p>The 'Type' column for the 'In' unit is circled in red.</p>	Product	Dereservation Date	Unit	Blood Group	Expiry Date	Type	State	Red Cells	28-Apr-2022 11:51:00	G052 522 720 729 G	A Neg	03-May-2022	Crossmatched	Out	Red Cells	28-Apr-2022 11:51:00	G052 522 700 641 M	A Pos	20-May-2022	Crossmatched	In
Product	Dereservation Date	Unit	Blood Group	Expiry Date	Type	State																
Red Cells	28-Apr-2022 11:51:00	G052 522 720 729 G	A Neg	03-May-2022	Crossmatched	Out																
Red Cells	28-Apr-2022 11:51:00	G052 522 700 641 M	A Pos	20-May-2022	Crossmatched	In																

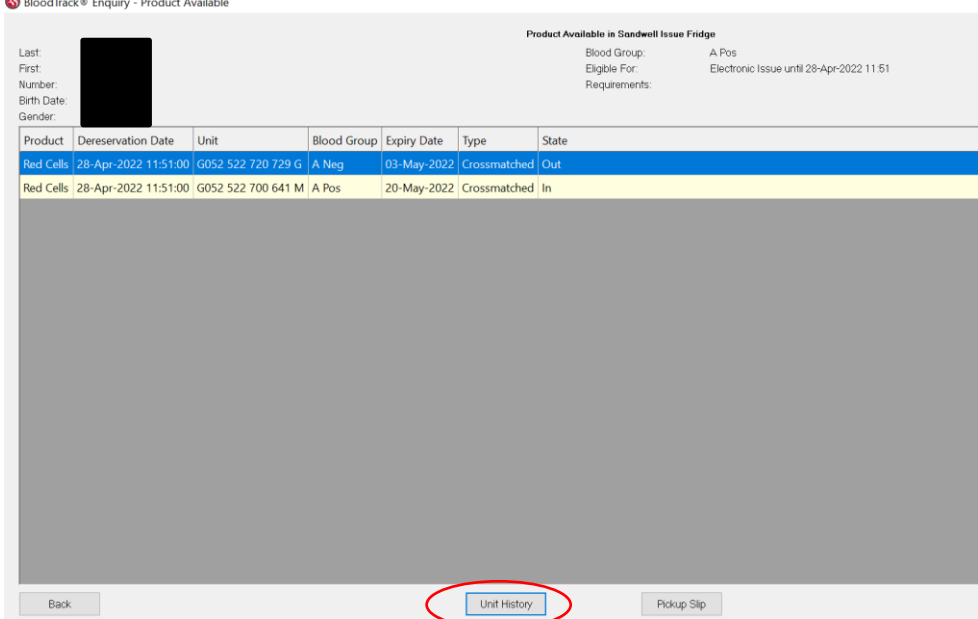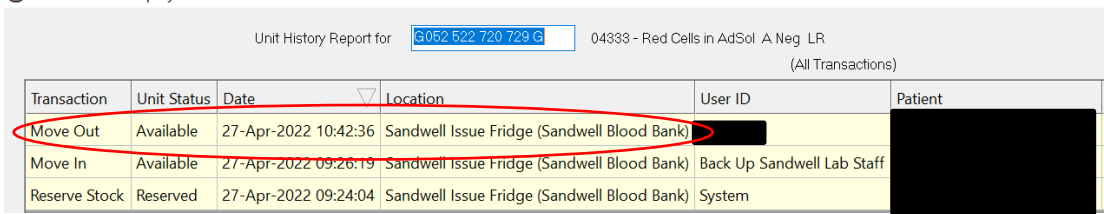

Step	Action																								
9	<p>You can also view the units history by highlighting the relevant unit and then click 'Unit History'</p>  <p>BloodTrack® Enquiry - Product Available</p> <p>Product Available in Sandwell Issue Fridge</p> <p>Last: [Redacted] First: [Redacted] Number: [Redacted] Birth Date: [Redacted] Gender: [Redacted]</p> <p>Blood Group: A Pos Eligible For: Electronic Issue until 28-Apr-2022 11:51 Requirements:</p> <table border="1"> <thead> <tr> <th>Product</th> <th>Dereservation Date</th> <th>Unit</th> <th>Blood Group</th> <th>Expiry Date</th> <th>Type</th> <th>State</th> </tr> </thead> <tbody> <tr> <td>Red Cells</td> <td>28-Apr-2022 11:51:00</td> <td>G052 522 720 729 G</td> <td>A Neg</td> <td>03-May-2022</td> <td>Crossmatched</td> <td>Out</td> </tr> <tr> <td>Red Cells</td> <td>28-Apr-2022 11:51:00</td> <td>G052 522 700 641 M</td> <td>A Pos</td> <td>20-May-2022</td> <td>Crossmatched</td> <td>In</td> </tr> </tbody> </table> <p>Buttons: Back, Unit History, Pickup Slip</p>	Product	Dereservation Date	Unit	Blood Group	Expiry Date	Type	State	Red Cells	28-Apr-2022 11:51:00	G052 522 720 729 G	A Neg	03-May-2022	Crossmatched	Out	Red Cells	28-Apr-2022 11:51:00	G052 522 700 641 M	A Pos	20-May-2022	Crossmatched	In			
Product	Dereservation Date	Unit	Blood Group	Expiry Date	Type	State																			
Red Cells	28-Apr-2022 11:51:00	G052 522 720 729 G	A Neg	03-May-2022	Crossmatched	Out																			
Red Cells	28-Apr-2022 11:51:00	G052 522 700 641 M	A Pos	20-May-2022	Crossmatched	In																			
10	<p>Here you can view who collected the unit and at what time it was removed from its issue location</p>  <p>BloodTrack® Enquiry - Product Available</p> <p>Unit History Report for G052 522 720 729 G 04333 - Red Cells in AdSol A Neg LR (All Transactions)</p> <table border="1"> <thead> <tr> <th>Transaction</th> <th>Unit Status</th> <th>Date</th> <th>Location</th> <th>User ID</th> <th>Patient</th> </tr> </thead> <tbody> <tr> <td>Move Out</td> <td>Available</td> <td>27-Apr-2022 10:42:36</td> <td>Sandwell Issue Fridge (Sandwell Blood Bank)</td> <td>[Redacted]</td> <td>[Redacted]</td> </tr> <tr> <td>Move In</td> <td>Available</td> <td>27-Apr-2022 09:26:19</td> <td>Sandwell Issue Fridge (Sandwell Blood Bank)</td> <td>Back Up Sandwell Lab Staff</td> <td>[Redacted]</td> </tr> <tr> <td>Reserve Stock</td> <td>Reserved</td> <td>27-Apr-2022 09:24:04</td> <td>Sandwell Issue Fridge (Sandwell Blood Bank)</td> <td>System</td> <td>[Redacted]</td> </tr> </tbody> </table>	Transaction	Unit Status	Date	Location	User ID	Patient	Move Out	Available	27-Apr-2022 10:42:36	Sandwell Issue Fridge (Sandwell Blood Bank)	[Redacted]	[Redacted]	Move In	Available	27-Apr-2022 09:26:19	Sandwell Issue Fridge (Sandwell Blood Bank)	Back Up Sandwell Lab Staff	[Redacted]	Reserve Stock	Reserved	27-Apr-2022 09:24:04	Sandwell Issue Fridge (Sandwell Blood Bank)	System	[Redacted]
Transaction	Unit Status	Date	Location	User ID	Patient																				
Move Out	Available	27-Apr-2022 10:42:36	Sandwell Issue Fridge (Sandwell Blood Bank)	[Redacted]	[Redacted]																				
Move In	Available	27-Apr-2022 09:26:19	Sandwell Issue Fridge (Sandwell Blood Bank)	Back Up Sandwell Lab Staff	[Redacted]																				
Reserve Stock	Reserved	27-Apr-2022 09:24:04	Sandwell Issue Fridge (Sandwell Blood Bank)	System	[Redacted]																				
11	<p>You can also see that the patient is eligible for electronic issue until the date and time shown</p>  <p>BloodTrack® Enquiry - Product Available</p> <p>Last: [Redacted] First: [Redacted] Number: [Redacted] Birth Date: [Redacted] Gender: [Redacted]</p> <p>Product Available Blood Group: A Pos Eligible For: Electronic Issue until 28-Apr-2022 11:51 Requirements:</p> <table border="1"> <thead> <tr> <th>Location</th> <th>Prod...</th> <th>Crossmatched</th> <th>Unallocated</th> </tr> </thead> <tbody> <tr> <td>Sandwell Issue Fridge</td> <td>Red Cells</td> <td>2</td> <td>0</td> </tr> </tbody> </table> <p>Therefore if more blood components were required for this patient before this date then the Blood Transfusion Lab would only require a request form and not another sample.</p>	Location	Prod...	Crossmatched	Unallocated	Sandwell Issue Fridge	Red Cells	2	0																
Location	Prod...	Crossmatched	Unallocated																						
Sandwell Issue Fridge	Red Cells	2	0																						
12	<p>If the following is shown then please contact the Blood Transfusion Lab to see if a sample is required</p>  <p>Product Available</p> <p>Blood Group: A Pos Eligible For: Not Eligible for Electronic Issue Requirements:</p> <p>*(Sample details can also be seen in UNITY)</p>																								

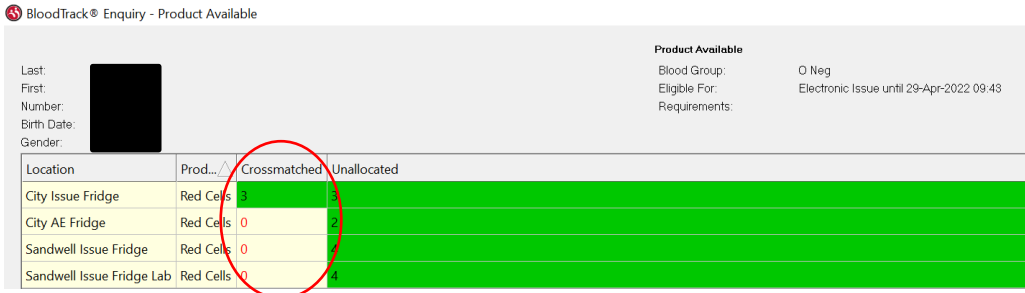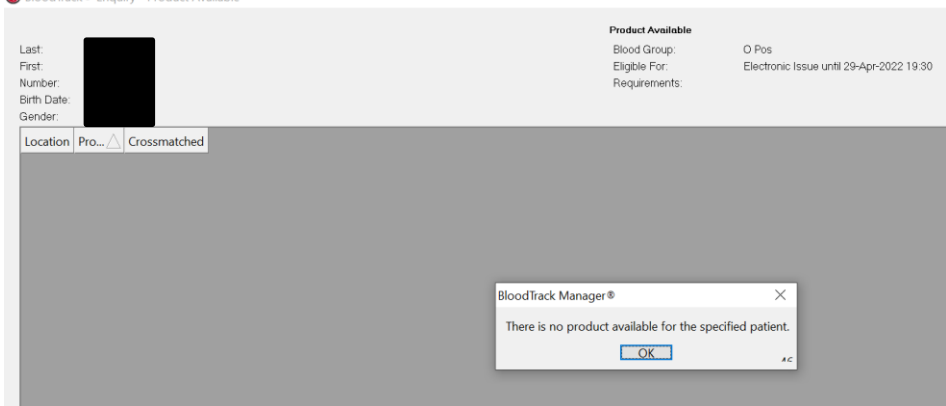

Step	Action																				
13	<p>If your patient is Blood Group O Neg then it will show all O Negs available as these are kept for emergency issue. You only need to look at the 'Crossmatched' column for what is specifically available for your patient</p> <p></p> <p>BloodTrack® Enquiry - Product Available</p> <p>Last: [Redacted] First: [Redacted] Number: [Redacted] Birth Date: [Redacted] Gender: [Redacted]</p> <p>Product Available Blood Group: O Neg Eligible For: Electronic Issue until 29-Apr-2022 09:43 Requirements:</p> <table border="1"><thead><tr><th>Location</th><th>Prod... / Δ</th><th>Crossmatched</th><th>Unallocated</th></tr></thead><tbody><tr><td>City Issue Fridge</td><td>Red Cells</td><td>3</td><td>3</td></tr><tr><td>City AE Fridge</td><td>Red Cells</td><td>0</td><td>0</td></tr><tr><td>Sandwell Issue Fridge</td><td>Red Cells</td><td>0</td><td>0</td></tr><tr><td>Sandwell Issue Fridge Lab</td><td>Red Cells</td><td>0</td><td>0</td></tr></tbody></table>	Location	Prod... / Δ	Crossmatched	Unallocated	City Issue Fridge	Red Cells	3	3	City AE Fridge	Red Cells	0	0	Sandwell Issue Fridge	Red Cells	0	0	Sandwell Issue Fridge Lab	Red Cells	0	0
Location	Prod... / Δ	Crossmatched	Unallocated																		
City Issue Fridge	Red Cells	3	3																		
City AE Fridge	Red Cells	0	0																		
Sandwell Issue Fridge	Red Cells	0	0																		
Sandwell Issue Fridge Lab	Red Cells	0	0																		
14	<p>If no blood is available for your patient the following will display</p> <p></p> <p>BloodTrack® Enquiry - Product Available</p> <p>Last: [Redacted] First: [Redacted] Number: [Redacted] Birth Date: [Redacted] Gender: [Redacted]</p> <p>Product Available Blood Group: O Pos Eligible For: Electronic Issue until 29-Apr-2022 19:30 Requirements:</p> <table border="1"><thead><tr><th>Location</th><th>Pro... / Δ</th><th>Crossmatched</th></tr></thead><tbody></tbody></table> <p>BloodTrack Manager® There is no product available for the specified patient. OK</p>	Location	Pro... / Δ	Crossmatched																	
Location	Pro... / Δ	Crossmatched																			

Please check on UNITY to ensure a request has been made