


## Urine Monovette® – Points to remember

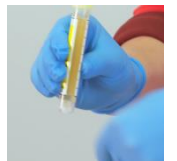

1. Collect urine into a clean collection cup/pot and sit on a flat surface.
2. Remove stopper and keep (**do not throw away**) .
3. Attach straw by pushing onto the Luer end of the Monovette.
4. Holding the tube, place the tip of the straw into the urine and begin to pull back on the plunger.
5. Draw the plunger only to the line, remove the tip from the urine and turn the tube upside down so that the tip is facing upwards and the plunger faces the floor.
6. Pull the plunger back to the end click into place which will remove the excess urine from the straw and prevent spillage.
7. Remove and discard of straw.
8. **Replace stopper.**
9. Snap off the plunger and discard.
10. Label and place into specimen bag.



**Natalka Prychidnyj**  
Product Support Specialist  
07818 542174  
natalka.prychidnyj@sarstedt.com

**Nouman Javid**  
Account Manager  
07970 220114  
nouman.javid@sarstedt.com