


Urine Monovette® User Guide


1

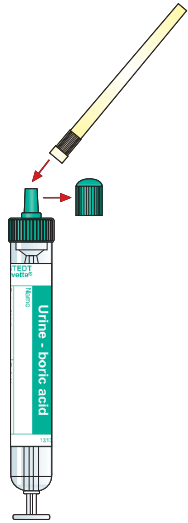

Remove stopper and keep for later use!


Attach plastic straw.

2

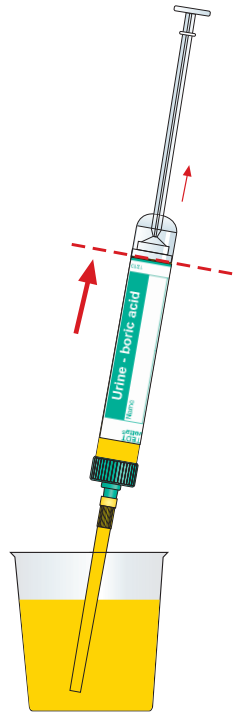


Insert the plastic straw into the container and fill the Urine Monovette® up to the bottom of the label.

3

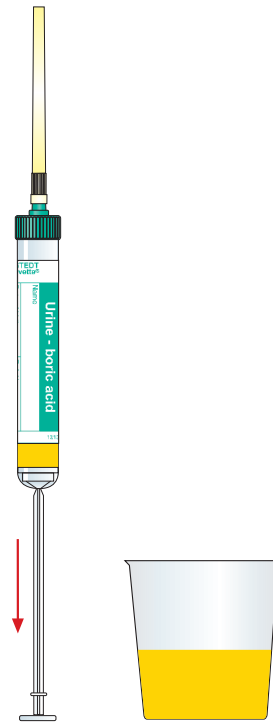

To empty the plastic straw, hold the Urine Monovette® in an upright position and pull the plunger backwards to the bottom of the tube.


4

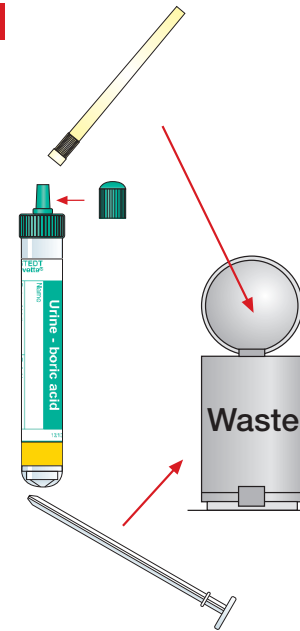

Remove the plastic straw, break off plunger and throw away.

Replace the stopper.

5

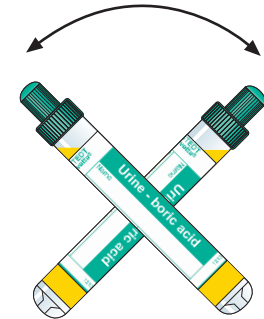

Mix well after sample collection (tilt approx. 5 times).

SARSTEDT Ltd.
68 Boston Rd,
Leicester LE4 1AW
Tel.: 0116 235 9023
Fax: 0116 236 6099
info@sarstedt.co.uk
www.sarstedt.com